COMPOSITION

Sels d'EDTA (20%), agent gélifiant.

PROPRIETES

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EDTA salts (20 %), gel-forming agent

PROPERTIES

Main component is EDTA salts. After injection into dentine, EDTA forms a complex with dentine calcium, loosens the structure of hard tooth tissues and makes easier the enlargement of calcified and sclerous orifice of the canals, cleaning and formation of hard accessible tooth canals. Gelling agent is a good lubricant for endodontic instruments. Water soluble gel base allow easy and fast washing of the canals.

INDICATIONS

EDTECT gel is used for chemical-mechanical enlargement, formation and antiseptic treatment of hard accessible root canals.

CONTRAINDICATIONS

Drug idiosyncrasy.
Use with great care for allergic patient.
Don't misuse.

SIDE EFFECTS

As far as all terms of storing, transporting and application are observed, there are no side effects.

METHOD OF USE

Attention! It is recommended to store material at room temperature (22–24) °C within 1–2 hours before use.

Before applying of **EDTECT gel**, isolate tooth with rubber dam.

Enlarge tooth cavity and clean the pulp chamber. Smear the instrument with **EDTECT gel** and fulfill mechanical enlargement without effort. Repeat procedure if necessary using fresh portion of gel.

Root canals enlargement is recommended to alternate with rinsing and irrigation by 1–3 % solution of sodium hypochlorite.

Attention! Do not leave material in the canal!

PRESENTATION

Gel (syringe)	5 ml / 10 ml
Tips	3 pc

STORAGE

Store at the temperature from 5 °C to 25 °C. Keep in dry place

Attention! Close materials tightly after use.

Do not use after expiry date

Shelf life — 3 years

PRECAUTIONS

Wear suitable protective clothing, gloves and goggles before use.

In case of eye or skin contact, as well as contact with soft tissues and mucosa of oral cavity, remove the material immediately and wash with water.

Use with caution in patients with mental disorders.

Do not use material for pregnant women (because of insufficient clinical data).

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TehnoDent Co. Ltd.