COMPOSITION

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Formaldehyde, phenol, filler

PROPERTIES

Phenol and formaldehyde are antiseptics denaturalizing microorganisms' proteins, thus providing antiseptic effect, deep mummification and root canals sterilization.

INDICATIONS

CRESOTIN Liquid No. 2 is used:

- for antiseptic treatment of infected root canals in complicated pulpitis
- resection of upper part of tooth root

CONTRAINDICATIONS

Individual intolerance to components of dental material.
Allergy to phenol and its derivatives.

SIDE EFFECTS

As far as all terms of storing, transporting and application are observed, there are no side effects.

METHOD OF USE

Attention! It is recommended to store material at room temperature (22–24) °C within 1–2 hours before use.

CRESOTIN Liquid No. 2 should be applied with the use of saturated and carefully squeezed cotton swabs, avoiding any leakage of the liquid, and left in tooth canal orifice for 2–3 days.

Saturated with **CRESOTIN Liquid No. 2** cotton swabs can be used for treatment of infected root canals or as a temporary application.

In 2–3 days take out cotton swab, treat the canal one more time and after that fill the sterile root canal with **CRESOTIN** paste, or any other paste based on chlorphenol and eugenol. Otherwise after taking out the cotton swab, provide canal treatment with liquid for dehydrating and degreasing, based on ethers and alcohol, and fill the canal with any sealer.

PRESENTATION

Liquid (bottle) 15 ml

STORAGE

Store at the temperature from 5 °C to 25 °C. Keep in dry place

Attention! Close materials tightly after use.

Do not use after expiry date

Shelf life — 3 years

PRECAUTIONS

Wear suitable protective clothing, gloves and goggles before use.

Avoid contact with eyes. In case of eye or skin contact, as well as contact with soft tissues and mucosa of oral cavity, remove the material immediately and wash with water.

Use with caution in patients with mental disorders.

Do not use material for pregnant women (because of insufficient clinical data).

In cases of allergic reactions in particularly sensitive patients, the material should be removed and its further application abandoned.

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TehnoDent Co. Ltd,
TEHNODENT GmbH