COMPOSITION

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Sodium and potassium salts of EDTA (20 %)

PROPERTIES

Main component is EDTA, after injection into dentine forms a complex with dentine calcium, loosens the structure of hard tooth tissues and makes easier an enlargement of calcified and sclerous orifice of the canals, cleans and forms hard accessible tooth canals.

INDICATIONS

- Detection of root canal orifice
- Chemical enlargement of root canals during preparing for filling

CONTRAINDICATIONS

Drug idiosyncrasy
Use with great care for allergic patient
Don't misuse

SIDE EFFECTS

No known side effects as far as the application of the product is concerned

METHOD OF USE

Attention! It is recommended to store material at room temperature (22–24) °C within 1–2 hours before use.

Liquid is applied with the help of pipette, root needle, cotton swab or mouth pack.

Put it into the pulp cavity to start mechanical enlargement.

Then using root needle inject the liquid into root canal for 2–3 minutes, repeat the procedure alternating chemical, mechanical enlargement with impregnation of dentine with fresh portion of the liquid if necessary.

PRESENTATION

Liquid (bottle) 15ml / 100ml

STORAGE

Store at the temperature from 5 °C to 25 °C. Keep in dry place

Attention! Close materials tightly after use.

Do not use after expiry date

Shelf life — 3 years

PRECAUTIONS

Wear suitable protective clothing, gloves and goggles before use.

In case of eye or skin contact, as well as contact with soft tissues and mucosa of oral cavity, remove the material immediately and wash with water.

Use with caution in patients with mental disorders.

Do not use material for pregnant women (because of insufficient clinical data).

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TehnoDent Co. Ltd,
**TEHNODENT GmbH**