**EN** INSTRUCTION FOR USE  
**DE** GEBRAUCHSANWEISUNG  
**FR** GUIDE D'UTILISATION

## EPOXIDIN *duo*

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CE 0653

**COMPOSITION**

Epoxy resin, amine hardening agents, zirconium oxide, filling component, plasticizing agent

**PROPERTIES**

Sealer EPOXIDIN duo is produced as dual syringe with white paste and yellow paste. Mixing of white and yellow pastes, in equal proportions, forms plastic paste with film width of 5–10 µm. Paste easily inserts into canal on gutta-percha points and isolates tooth root canals well. At room temperature (20–23) °C paste keeps its plasticity during 4 hours. Paste hardens in root canal within 24 hours and forms bioinert amino resin with good adhesion and low solubility. Overfilling material in apex area will not lead to any complications but can cause discomfort for a patient in treated tooth.

**INDICATIONS**

Sealer EPOXIDIN duo is used for filling of tooth root canals with gutta-percha points, silver points and titanium points

**CONTRAINDICATIONS**

Drug idiosyncrasy  
Allergy to amine-containing epoxy resins  
Do not misuse

**SIDE EFFECTS**

As far as all terms of storing, transporting and application are observed, there are no side effects.

**METHOD OF USE**

**Attention!** It is recommended to store material at room temperature (22–24) °C within 2 hours before use.

Dual syringe allows to push out both pastes in equal volumes simultaneously and with the help of tip mix them until homogeneous paste is obtained, which can be inserted directly into the root canal through the cannula.

Insert the paste into sterile and dried canal with small portions using root needle or root canal filling instrument. Inserting and removing of paste with root canal filling instrument should be performed at low speed of rotation to avoid air bubbles in material. During filling with gutta-percha points, insert small amount of material into the canal and seal with points doped in paste. For condensation and obturation with hot gutta-percha it is recommended to use small amount of sealer for covering only the walls of canal. Do not overfill the canal with material to avoid entering of out-of-apex area. In case of inserting material into inferior dental canal, remove it immediately out of the canal. After filling of the canal make isolation and tooth crown restoration. If it is necessary to remove material out of the canal, filled with sealer and gutta-percha, it is recommended to use thermomechanical treatment.

**Attention!** Clean the instruments with solvents after work (alcohol or acetone).

**PRESENTATION**

Dual syringe with white and yellow pastes	12 g
Mixing tips & Intraoral tips	1 / 10 pcs or Spatule 1 pc
Mixing pad	1 pc

**STORAGE**

Store at the temperature from 5 °C to 25 °C. Storage at higher temperature leads to paste splitting. However, it does not affect setting time.

**Attention!** Close materials tightly after use.

Do not use after expiry date

Shelf life — 3 years

**PRECAUTIONS**

Wear suitable protective clothing, gloves and goggles before use.

Avoid contact with eyes. In case of eye or skin contact, as well as contact with soft tissues and mucosa of oral cavity, remove the material immediately and wash with water.

Use with caution in patients with mental disorders.

Do not use material for pregnant women (because of insufficient clinical data).

In cases of allergic reactions in particularly sensitive patients, the material should be removed and its further application abandoned.

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DUAL-SYRINGE USAGE RECOMMENDATIONS	
1  Remove the cap by turning it 90° clockwise.	2  To ensure an appropriate mixing, prior to the first use, dispense small amount of paste from the syringe.
3  Attach the mixing tip to the syringe, aligning the notch. Turn the tip 90° clockwise to secure it in place.	4  Adjust the intraoral tip and rotate it in angle for proper injection of paste into root canal.
5  With equal pressure on the plunger push needed amount of paste into the root canal.	6  After application remove mixing tip with intraoral tip by turning it 90° clockwise.
7  Put on the cap on the syringe head, aligning the notch. Turn it 90° clockwise.	

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TEHDODENT GmbH