

INSTRUCTION FOR USE CRESOTIN

Liquid No. 1

COMPOSITION

Lidocaine hydrochloride, camphor, chlorophenol, eugenol, m-cresol, filler

PROPERTIES

Lidocaine hydrochloride provides access to carious cavity and neutralizes pain expansion in nerve fibers, thus protecting the pulp and enable penetration of all components in hard tooth tissues. Eugenol possessing sedative properties, decrease aggressive influence on the pulp and soft tissues. Chlorophenol, eugenol and m-cresol are broad spectrum antiseptics, possessing slightly mummifying effect on nerve fibers. Camphor mitigates the impact of chlorophenol on the body.

INDICATIONS

Liquid No. 1 CRESOTIN is used:

- for anesthesia in traumatic and acute pulpitis before applying of devitalization paste (DEVITAL, TehnoDent)
- in periodontitis
- as a long-acting application for pulpitis management

CONTRAINDICATIONS

Drug idiosyncrasy. Do not misuse.

SIDE EFFECTS

As far as all terms of storing, transporting and application are observed, there are no side effects.

METHOD OF USE

Attention! It is recommended to store material at room temperature (22–24) °C within 1–2 hours before use.

Liquid No. 1 CRESOTIN should be applied with cotton swab or cotton ball for applications, saturated with liquid and slightly squeezed in order to avoid leakage of liquid.

During treatment of deep caries and acute pulpitis insert saturated swab in the cavity, isolate with temporary material (TEMPODENT, TehnoDent) without compressing and leave for 1–3 days.

During long-term treatment, using **liquid No. 1 CRESOTIN** as a long-acting unpenetrable application, put saturated cotton ball under a temporary filling (TEMPODENT, TehnoDent). In a period (up to 10 days), determined by dentist, planned treatment procedures can be fulfilled.

PRESENTATION

Liquid (bottle) 15 ml

STORAGE

Store at the temperature from 5 °C to 25 °C. Keep in dry place.

Tightly close container immediately after use.

Do not use after expiry date.

Shelf life — 3 years.

MANUFACTURER

TehnoDent Co. Ltd

1/6 Beryozovaya str., Severnyi,

Belgorod region, Russia, 308570

Tel. +7 (495) 118-28-73

E-mail: info@tehnodent.org

www.tehnodent.org

For professional use only

1.015.002-01_EN_002

Rev.: 2019-08