COMPOSITION

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COMPOSITION

Epoxy resin, amine hardening agents, zirconium oxide, filling component, plasticizing agent

PROPERTIES

Sealer **EPOXIDIN** is produced as a dental kit: white paste and yellow paste. Mixing of white and yellow pastes, in equal proportions, forms plastic paste with film width of 5–10 µm. Paste easily inserts into canal on gutta-percha points and isolates tooth root canals well. At room temperature (20–23) °C paste keeps its plasticity during 4 hours. Paste hardens in root canal within 24 hours and forms bioinert amino resin with good adhesion and low solubility. Overfilling material in apex area will not lead to any complications but can cause discomfort for a patient in treated tooth.

INDICATIONS

Sealer **EPOXIDIN** is used for filling of tooth root canals with gutta-percha points, silver points and titanium points

CONTRAINDICATIONS

Drug idiosyncrasy
Do not misuse

SIDE EFFECTS

As far as all terms of storing, transporting and application are observed, there are no side effects

METHODS OF USE

Attention! It is recommended to store material at room temperature (22–24) °C within 2 hours before use.
Mix white and yellow pastes on glass plate or mixing pad with dry clean spatula at the volume ratio 1:1 till homogeneous consistency and color.

Attention! Do not press too hard when closing the tube. Clean tube nozzle before closing. Tightly close the tubes without changing of caps: close orange tube with orange cap and white tube with white cap. Noncompliance of recommendations leads to pastes hardening in tubes and insufficiency in application.
Attention! Insert the paste into sterile and dried canal with small portions using root needle or root canal filling instrument. Inserting and removing of paste with root canal filling instrument should be performed at low speed of rotation to avoid air bubbles in material. During filling with gutta-percha points, insert small amount of material into the canal and seal with points doped in paste. For condensation and obturation with hot gutta-percha it is recommended to use small amount of sealer for covering only the walls of canal. Do not overfill the canal with material to avoid entering of out-of-apex area. In case of inserting material into inferior dental canal, remove it immediately out of the canal. After filling of the canal make isolation and tooth crown restoration. If it is necessary to remove material out of the canal, filled with sealer and gutta-percha, it is recommended to use thermomechanical treatment.

Attention! Clean the instruments and mixing pad with solvents after work (alcohol or acetone).

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STORAGE

Store at the temperature from 5 °C to 25 °C. Storage at higher temperature leads to paste splitting. However, it does not affect setting time.

Attention! Close materials tightly after use.

Do not use after expiry date
Shelf life — 3 years

PRECAUTIONS

Wear suitable protective clothing, gloves and goggles before use.
Avoid contact with eyes. In case of eye or skin contact, as well as contact with soft tissues and mucosa of oral cavity, remove the material immediately and wash with water. Use with caution in patients with mental disorders.
Do not use material for pregnant women (because of insufficient clinical data).
In cases of allergic reactions in particularly sensitive patients, the material should be removed and its further application abandoned.

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TehnoDent Co. Ltd,
TEHNODENT GmbH