COMPOSITION

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Stabilized sodium hypochlorite (concentration — 3 %, 5,2 %)

PROPERTIES

Effective cleaning of root canal system.
Sodium hypochlorite (5.2 %) concentrated can dissolve vital pulp remains.

INDICATIONS

- treatment of infected root canals with pulpitis and periodontitis
- treatment of prepared tooth cavity before filling

CONTRAINDICATIONS

Drug idiosyncrasy
Do not misuse
Use with great care with allergic patients

SIDE EFFECTS

As far as all terms of storing, transporting and application are observed, there are no side effects

METHODS OF USE

Attention! It is recommended to store material at room temperature (22–24) °C within 1–2 hours before use.

Liquid is used with the help of dropping pipette, root needle, cotton swab or by tamponing. Effective cleaning of root canal system is provided during two-step treatment with **ANTISEPTIC LIQUID No. 2** (3 %) and EDTA solution, using endodontic needle with side hole (or bulb syringe) for injecting solutions into deep areas of root canals. It is necessary to use salivary ejector to avoid burn injury of oral mucosa because of high alkalinity of the liquid. At final irrigation leave 15–20 ml of sodium hypochlorite (3 %) in the canal for 5–7 min., then wash with distilled water, dry with sterile paper point or useliquid for dehydrating and degreasing.

Concentrated liquid of sodium hypochlorite (5.2 %) should be diluted with distilled water to reach needed concentration. Without diluting liquid is able to dissolve vital pulp remains.

PRESENTATION

Liquid (bottle) 100 ml / 300 ml

STORAGE

Store at the temperature from 5 °C to 25 °C. Keep in dry place

Attention! Close materials tightly after use.

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TehnoDent Co. Ltd.