

INSTRUCTION FOR USE
CALCETIN
Cavity liner

Powder/Paste

COMPOSITION

Calcium hydroxide, barium sulfate, paste-forming agent.

PROPERTIES

Calcium Hydroxide form an alkaline medium (pH more than 12) which provides durable antibacterial effect, increases pulp regeneration abilities and initiate formation of secondary dentine.

INDICATIONS

- Direct and indirect pulp capping
- Traumatic, acute serous and chronic fibrous pulpitis
- Apexification and apexogenesis

CONTRAINDICATIONS

Drug idiosyncrasy.
Use with great care for allergic patient.
Do not misuse.

SIDE EFFECTS

As far as all terms of storing, transporting and application are observed, there are no side effects

METHOD OF USE

Attention! Mix one dose meter (0.08–0.10) g of powder with 2–3 drops (0.08–0.1) g of distilled water on a mixing pad or glass plate for approx 1 minute. At room temperature (18–23) °C and relative humidity (45–55) % **CALCETIN** remains workable for approx 6–8 minutes.

For treatment of deep caries apply **CALCETIN** at the bottom of dried cavity. Remove the excess of paste. Dry the paste with an air jet. Apply cement and start tooth crown restoration.

For pulp regeneration apply the paste after formation of deep cavity or in case of traumatic opening of the pulp. Treat the cavity with antiseptic and dry carefully. Apply **CALCETIN** up to dentino-enamel level, dry with an air jet. Tightly fill with GI cement.

In 1–2 weeks if no pain left, temporary filling can be removed completely or partially if needed. After cleaning of cavity and its antiseptic treatment, apply calcium lining. Then use GI cement and make tooth crown restoration. Re-examine the patient in 3–6 months time.

PRESENTATION

Paste (syringe)	7 g
Powder (jar)	7 g

STORAGE

Store at temperature 5 °C to 25 °C, in a well sealed container
Shelf life — 3 years

Attention! Close the materials tightly after use
The material must not be used after the expiry date
Shelf life — 3 years

MANUFACTURER

TehnoDent Co. Ltd
1/6 Beryozovaya str., Severnyi,
Belgorod region, Russia, 308570
Tel. +7 (495) 118-28-73
E-mail: info@tehnodent.org
www.tehnodent.org

For professional use only

1.004.002-01_EN_002

Rev.: 2019-08