COMPOSITION

EDTA (17 %), peroxyde (10 %), agent gélifiant.

PROPRIETES

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EDETECT

Gel with peroxide

EN Gel for orifice detection and chemical and mechanical enlargement of root canals

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DE Hersteller

FR Fabricant

EN Authorized representative in the European Community

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EDTA (17 %), peroxide (10 %), gel-forming agent

PROPERTIES

Main component is EDTA. After injection into dentine, EDTA forms a complex with dentine calcium, loosens the structure of hard tooth tissues and makes easier the enlargement of calcified and sclerotic orifice of the canals, cleaning and formation of hard accessible tooth canals. Gelling agent is a good lubricant for endodontic instruments. Water soluble gel base allow easy and fast washing of the canals.

INDICATIONS

- Revealing and formation of root canal orifice
- Lubricating of dental instruments during chemical and mechanical enlargement of root canals

CONTRAINDICATIONS

Drug idiosyncrasy.
Use with great care for allergic patient.
Don't misuse.

SIDE EFFECTS

As far as all terms of storing, transporting and application are observed, there are no side effects.

METHOD OF USE

Attention! It is recommended to store material at room temperature (22–24) °C within 1–2 hours before use.

Before applying of **EDETECT** gel with peroxide isolate tooth with rubber dam. Enlarge tooth cavity and clean the pulp chamber. Wash root canal with 1–3 % solution of sodium hypochlorite, then put gel on endodontic instrument and with the help of reamer or K-files achieve active foam formation in root canal.

On half way of root canal length clean the canal contents and wash it with sodium hypochlorite solution using endodontic needle. Then lubricate endodontic instrument for enlargement with fresh portion of **EDETECT** gel with peroxide and fulfill mechanical enlargement of the canal.

If necessary repeat procedure several times using fresh portion of gel. Then wash the canal with sodium hypochlorite solution using endodontic needle until complete stop of foam formation.

Final washing of the canal should be done with distilled water.

Attention! Do not leave material in the canal.

PRESENTATION

Gel (syringe)	5 ml / 10 ml
Tips	3 pc

STORAGE

Store at 5 °C to 25 °C, in a well sealed container.

Attention! Close material tightly after use.

The material must not be used after the expiry date.

Shelf life — 3 years.

PRECAUTIONS

Wear suitable protective clothing, gloves and goggles before use.

In case of eye or skin contact, as well as contact with soft tissues and mucosa of oral cavity, remove the material immediately and wash with water.

Use with caution in patients with mental disorders.

Do not use material for pregnant women (because of insufficient clinical data).

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TehnoDent Co. Ltd,