




Poliren No.2, Ftor-lux kit

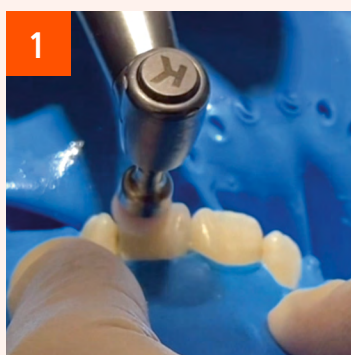

A 7-year-old child
Permanent teeth emerged with the enamel hypoplasia

- 1 — treat the teeth with Poliren No. 2 paste (fluoride-free) for polishing tooth enamel
- 2 — dry the tooth enamel thoroughly with an air flow, apply Ftor-lux Liquid for deep fluoridation (rub it into tooth enamel with careful massaging movements)
- 3 — after drying the tooth enamel with an air flow, apply the Ftor-lux Suspension for deep fluoridation, and rub it well into the tooth enamel
- 4 — tooth surface after deep fluoridation with Ftor-lux and rinsing the teeth with water

Recommendations:

professional oral hygiene procedures once every 3–4 months, repeat deep fluoridation after 1–2 weeks

We would like to thank Ekaterina Kuznetsova, Pediatric Dentist, qualification category II, Novosibirsk, for the materials provided